

Product Information

PAPANICOLAOU (PAP) STAIN FOR CYTOLOGIC SPECIMEN

(Progressive Method-Gill's Papanicolaou Template)

LEICA AutoStainer XL Stainer

Procedure HT40-GHS

| | | | | | | | | | | | | |
|-----------|-------------------------------|-------------------------------|----------------------------------|---------------------------------|---------------------------------|---------------------------------|-----------------------------|------------------------|------------------------|--------------------------------------|---------------------------------------|------|
| Station 1 | Station 2 | Station 3 | Station 4 | Station 5 | Station 6 | Station 7 | W1 | W2 | W3 | W4 | W5 | |
| Not Used | 100% Alc 30 sec | 100% Alc 30 sec | 95% Alc 15 sec | EA-50 5 min | Not Used | 80% Alc 2 min | Wash 2 min, 30 sec | Not Used | Wash 2 min | Wash 1 min | Wash 2 min | Oven |
| End | Station 18 Xylene 2 min | Station 17 Xylene 1 min | Station 16 100% Alc 15 sec | Station 15 95% Alc 15 sec | Station 14 95% Alc 10 sec | Station 13 95% Alc 10 sec | Station 12 OG-6 3 min | Station 11 Not Used | Station 10 Not Used | Station 9 Scott's Tap 1 min | Station 8 Gill's 2 Hem 4 min | Hold |