

Product Information

PAPANICOLAOU (PAP) STAIN FOR CYTOLOGIC SPECIMEN

(Regressive Method-Harris Papanicolaou Template)

LEICA AutoStainer XL Stainer

Procedure HT40-HHS

Station 1	Station 2	Station 3	Station 4	Station 5	Station 6	Station 7	W1	W2	W3	W4	W5	
Not Used	100% Alc 30 sec	100% Alc 30 sec	95% Alc 15 sec	EA-50 5 min	Not Used	80% Alc 2 min	Wash 2 min, 30 sec	Wash 2 min	Wash 2 min	Wash 1 min 30 sec	Wash 3 min	Oven
End	Station 18 Xylene 2 min	Station 17 Xylene 1 min	Station 16 100% Alc 15 sec	Station 15 95% Alc 15 sec	Station 14 95% Alc 10 sec	Station 13 95% Alc 10 sec	Station 12 OG-6 3 min	Station 11 Not Used	Station 10 Scott's Tap 1 min	Station 9 Diff Soln 10 sec	Station 8 Harris Hem 4 min	Hold